
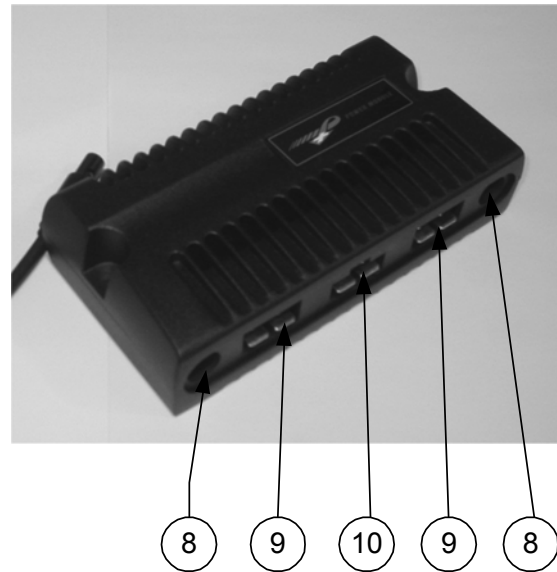


# ATTENDANT CONTROL ADAPTION USER MANUAL

This user manual is an appendix to the Enigma Energi Owner's Handbook. This user manual should be read in conjunction with the Enigma Energi Owner's Handbook. This appendix covers the operation and simple maintenance of the attendant control adaption for the Enigma Energi. The attendant control adaption is so that only an attendant walking behind the occupant may drive the powerchair.

## PARTS DESCRIPTION AND OPERATION

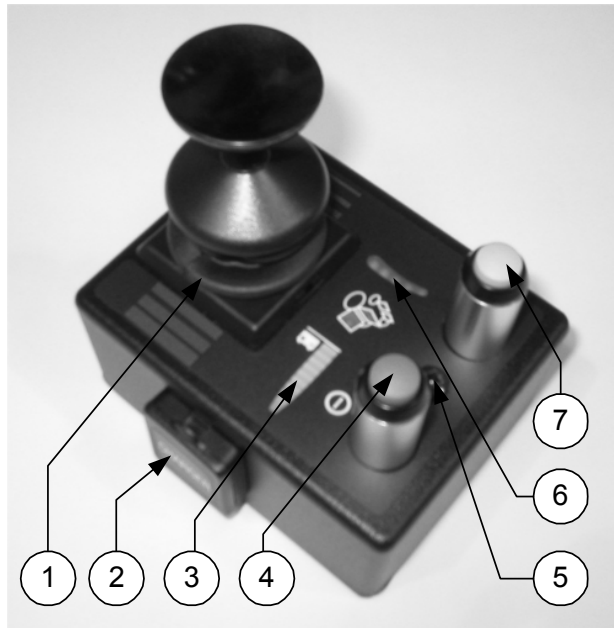

- 1. Joystick** Deflect the joystick in the direction of required travel. The further the joystick is deflected the greater the speed.
- 2. Charger Socket**
- 3. Power Gauge** Recharge the battery when only red lights are showing. When the gauge is flashing this indicates a fault with the powerchair.
- 4. On / Off button** Press to switch the powerchair on or off.
- 5. Height adjustment screw** Loosen the screw to adjust the height of the two buttons and retighten to secure.
- 6. Speed Indicator** Shows the maximum speed of the powerchair.
- 7. Speed selection button** Push the button and the speed indicator will flash. Push the joystick left or right to alter the maximum speed and press button again to set.
- 8. Bus lead port** Plug the bus lead from the attendant remote in to this port
- 9. Motor ports** Plug the two motors in to these two ports.
- 10. Battery port** Plug the battery box cable in to this port.


## **CONFIGURATION**

The attendant control powerchair uses a PG Drives Pilot+ as a central hub. The motors, battery boxes and Pilot+ ACU Remote control plug in to the Power Module.

Plug in the motors and battery boxes in to the Power Module.

Then plug the Pilot+ ACU Remote in to the Power Module using a Bus lead. Plug on end of the lead in to one of the control gear ports on the Power Module and the other end in to one of the Bus ports on the Pilot+ ACU Remote control. The Pilot+ ACU Remote should already be mounted to one of the back posts of the powerchair and should be comfortable for an Attendant to use whilst holding on to the two push handles.

If in any doubt please refer to the diagram below:

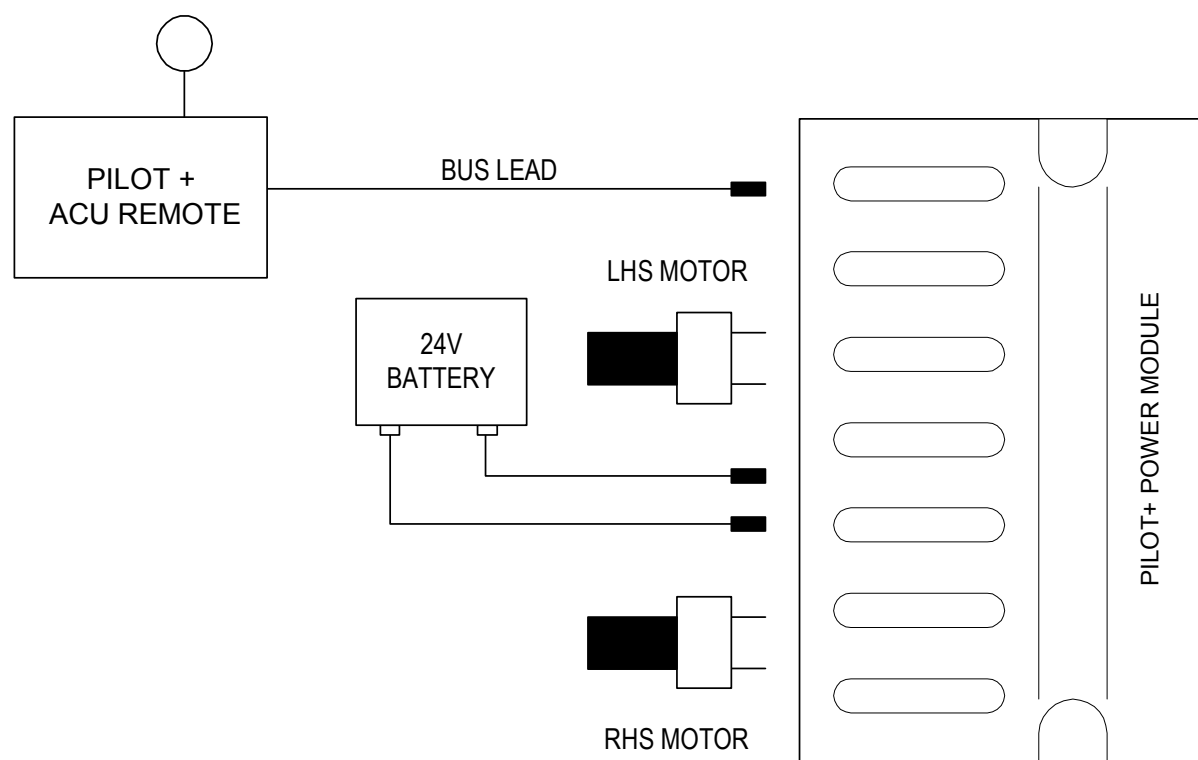

## **BATTERY CHARGING**

The battery charging point is located as shown on the photograph above. There is also a power gauge on the Pilot+ ACU Remote to show the state of the batteries. Please follow the battery charging instructions in the Energi Owner's Manual.